



02/9503020

TIPS ON TIPS:



NaviTip 29ga

File-Eze
EDTA
UltraCal XS



NaviTip 30ga


File-Eze
EDTA




NaviTip 31ga with
Double Sideport
Irrigator Tip

NaOCl
EDTA
File-Eze
CHX

БОРЕПИ на Riitano

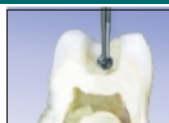

1.
2.

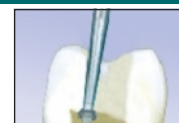

TILOS

FILL
NaOCl

FILL
File-Eze



FILL
NaOCl

FILL
File-Eze



FILL
NaOCl

FILL
File-Eze



FILL
NaOCl

FILL
File-Eze



FILL
NaOCl



FILL
NaOCl



FILL
NaOCl



FILL
NaOCl



FILL
NaOCl



SOAK
EDTA
30-90 sec.

SOAK
NaOCl
1 min.

EVAC

RINSE
STERILE
H₂O

SOAK
EDTA
30-60 sec.

SOAK
CHX
5 min.


EVAC

DAMP
1-2 pts.

FILL
NaOCl



Riitano Access Burs

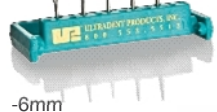

-6mm

3 (2 -4) WL-

PATENCY

GLIDE PATH

GLIDE PATH/APICAL

CROWN DOWN

APICAL FINISHING

- WL


Ultradent,

Consepsis ChlorCid

NaviTip #31



EndoREz®

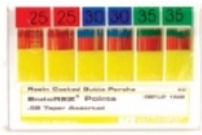

1. EndoRez,




2. EndoRez



3. EndoREz



4. NaviTip®,

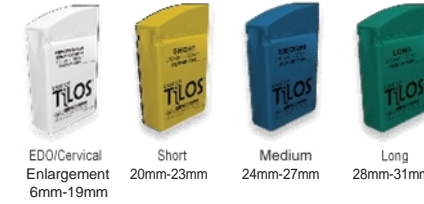


EDO/Cervical Enlargement 6mm-19mm, Short 20mm-23mm, Medium 24mm-27mm, Long 28mm-31mm

5. NaviTip Skini Syringe



6.

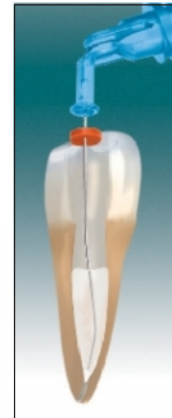

7. NaviTip 2-4mm

8. NaviTip

9.

EndoREz,

10.


11.

EndoREz Accelerator

12.

40

13.

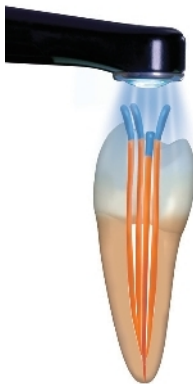

14.



15.

Ro gr.



Други ендодонтски



UniCore®

: 0.6, 0.8, 1.0, 1.2, 1.4



PermaFlo® Purple



OpalDam®



DermaDam®



DermaFrame

ULTRADENT®